

Introducing
2.5 NVG 
READY2CLIP
collection

In rural areas, people usually travel from far to get their eyes tested. The revolutionary **Ready2Clip™** collection allows vision care providers to deliver glasses customized for their prescription, gender and face shape on the spot, saving them a return trip.



DELIVERY ON THE SPOT



CUSTOMIZABLE




COMFORTABLE



EUROPEAN QUALITY STANDARDS

WHO CAN USE THE READY2CLIP™ COLLECTION?

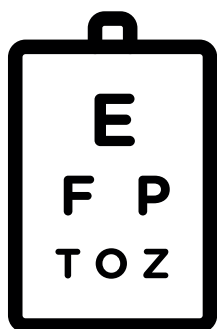

People suffering from **myopia, hyperopia or presbyopia**

No or slight



Pupillary distance between



52mm and 63mm

With same or different power in right and left eyes between -4.00 to +4.00



WHAT'S IN THE COLLECTION?

2.5 NVG READY2CLIP I

Offers on-the-spot customization with symmetrical pre-cut lenses that can be easily fitted into both the left and right sides of the frame.

It can fit both adults and children with pupillary distances of **58mm (children)** and **63mm (adults)**.



2.5 NVG READY2CLIP II

To enable a larger population to enjoy on-the-spot customization, we developed the Ready2Clip™ Generation II for Children. This series offers improved features:

1 Streamlined design and lighter frame
– greater comfort for wearers

2 Easy click release bridge
– easier for vision care providers to customize and assemble


3 Different-size bridges and adjustable temples – fits wearers with pupillary distances of 52mm – 60mm



MOUNTING GUIDELINES


FOR THE READY2CLIP™ GENERATION I

STEP 1

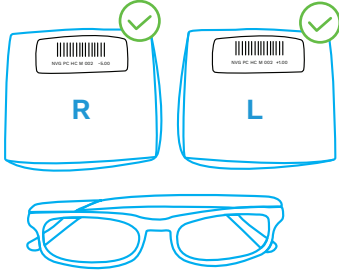

Select the **correct power** for right and left eye.


STEP 2

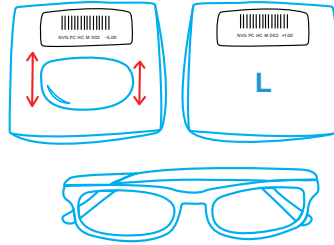

- Prepare for assembly.
- Start with **one lens at a time**.
- Leave the other lens in the envelope to avoid mix-up.


Fitting Tips: The lens is narrower at the nose side.

STEP 3

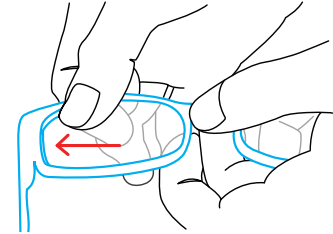


Remove both dummy lenses from the frame. Position the lens bevel in the groove, at the **temporal side**.

NOTE!

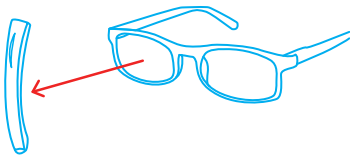


The **convex** side of a lens should always be in **front of the frame** when fitted.

STEP 4

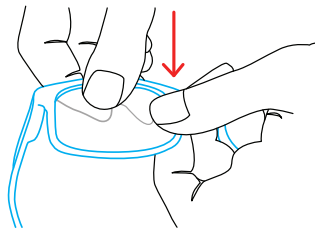


Maintain the lens in this position. Press down the lens by the nose side.

STEP 5

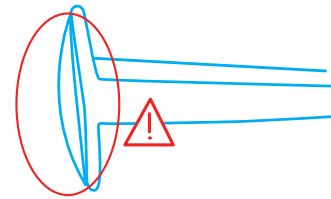


Check peripheral. Ensure there is **no protruding part** and that the lens is well fitted into the frame. Try shifting the lens to make sure there is no gap.

STEP 6

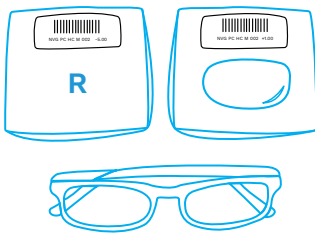

Repeat the same procedures for the other lens.

STEP 7

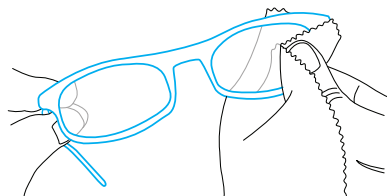

Once both lenses are fitted, wipe with a clean cloth.




Scan the QR code to view a video of how to mount the Ready2Clip™ Generation I

MOUNTING GUIDELINES


FOR THE READY2CLIP™ GENERATION II FOR CHILDREN

STEP 1

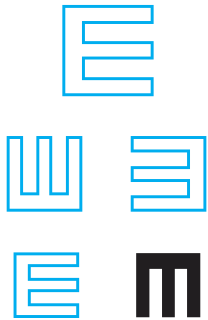

Determine the required correction.


STEP 2

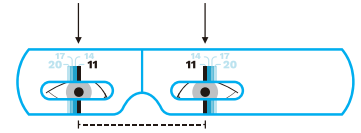

A. On the ruler, choose the above color for the target model to test.

B. Measure the PD, and record the bridge size.

STEP 3




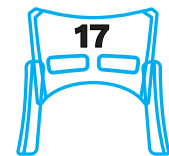


A. Choose the desired color for the frame.

B. Choose 2 pieces of lens for the required correction.

C. Choose the required size of bridge.

STEP 4

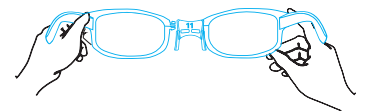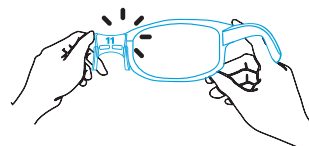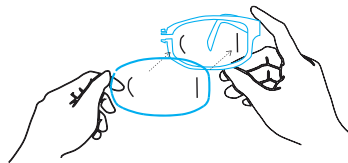
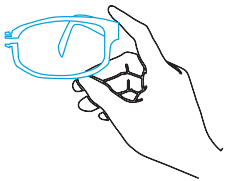


A. Hold the right part of the frame in your right hand.

B. Clip the required lens into the frame with the curved side nearer to the bridge.

C. Clip on the bridge and hear the click sound when done correctly.

D. Repeat step 4A - 4C for other lens to complete the process.

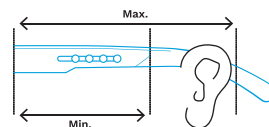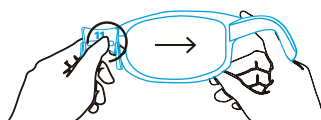

E. Test glasses on person. Check if the glasses fit.

F. Press the bridge button and pull the frame away only when necessary.

G. Clip into the right temple length according to the ear distance.

H. Clean and deliver!



Scan the QR code to view a video of how to mount the Ready2Clip™ Generation II for Children