

FOR CHILDREN

Introducing

2.5 NVG
READY2CLIP II

In rural areas, people usually travel from far to get their eyes tested. The revolutionary Ready2Clip™ collection allows vision care providers to deliver glasses customized for their prescription, gender and face shape on the spot, saving them a return trip.

Our latest range – **Ready2Clip™ Generation II for Children** – offers the same core benefit of on-the-spot customization with improved features:

Streamlined design and lighter frame


Greater comfort for wearers

Easy click release bridge

Easier for vision care providers to customize and assemble

Different-size bridges and adjustable temples

Fits wearers with pupillary distances of 52mm – 60mm

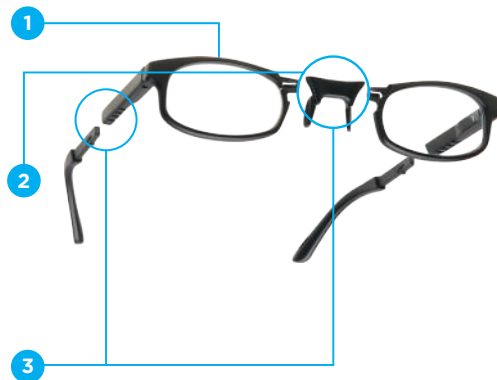

DELIVERY ON THE SPOT



CUSTOMIZABLE



COMFORTABLE



EUROPEAN QUALITY STANDARDS

WHO CAN USE THE READY2CLIP™ II FOR CHILDREN?



Children suffering from **myopia or hyperopia**



Pupillary distance between



With same or different power in right and left eyes between -4.00 to +4.00

PRODUCT OVERVIEW / SPECIFICATIONS

Specifications	Frame	Lenses
Material/ Feature	Polycarbonate Full Frame Spring Hinges Changeable Bridges Adjustable Temples	Hard Coat Polycarbonate Hard Coat Photochromic 1.56
Power		-4.00 to +4.00: 0.25 Step Only SPH, No CYL
PD		52-60mm



To browse the full range of Ready2Clip™ glasses, please visit: <https://2-5-nvg.myshopify.com/collections/ready-2-clip>

WHAT'S INSIDE YOUR READY2CLIP™ GENERATION II FOR CHILDREN KIT?

FRONT & TEMPLE



ADJUSTABLE TEMPLE SIZE

Mini 125 mm

Small 130 mm

Medium 135 mm

Large 140 mm



3 BRIDGE SIZES

Small 11 mm

Medium 14 mm

Large 17mm



CHOOSE THE COLOR OF YOUR KIT

R2C II 300 02 black

R2C II 300 05 blue

R2C II 300 01 brown



MOUNTING GUIDELINES

STEP 1




Determine the required correction.

STEP 2



A. On the ruler, choose the above color for the target model to test.

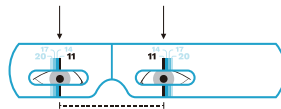

B. Measure the PD, and record the bridge size.

STEP 3



A. Choose the desired color for the frame.




B. Choose 2 pieces of lens for the required correction.




C. Choose the required size of bridge.


STEP 4

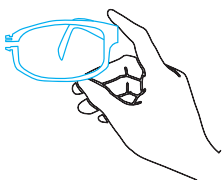


A. Hold the right part of the frame in your right hand.

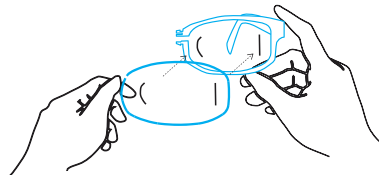


B. Clip the required lens into the frame with the curved side nearer to the bridge.

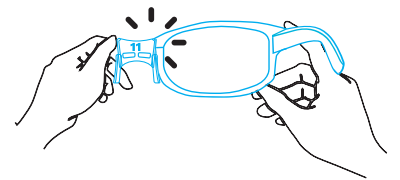


C. Clip on the bridge and hear the click sound when done correctly.

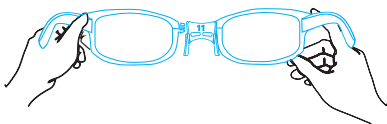


D. Repeat step 4A - 4C for other lens to complete the process.

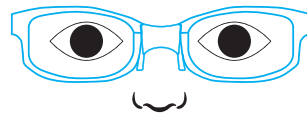

E. Test glasses on person. Check if the glasses fit.

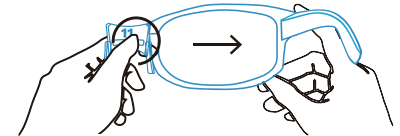

F. Press the bridge button and pull the frame away only when necessary.

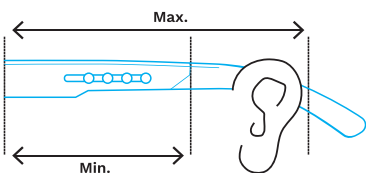

G. Clip into the right temple length according to the ear distance.



H. Clean and deliver!



Scan the QR code to view a video of how to mount the Ready2Clip™ Generation II for Children